Calcium Chloride 0.02M

PRESENTACION / CONTENTS

REF	3510401 Calcium Chloride 0.02M 3 x 16 mL
------------	--

Sólo para uso diagnóstico *in vitro*
For *in vitro* diagnostic use only

Calcium Chloride 0.02M

Determinación de factores de coagulación
Determination of coagulation factors

FUNDAMENTO

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REFERENCIAS

1. Triplett DA, Heparin: Clinical use and Laboratory Monitoring. In Triplett DA, Laboratory Evaluation of Coagulation, American Society of Clinical Pathologists Press, Chicago p 272 (1982).
2. Harker LA, Hemostasis Manual, FA Davis Co, Philadelphia p 62 (1974).
3. Triplett DA, Harms CS, Procedures for the Coagulation Laboratory, American Society of Clinical Pathologists Press, Chicago, p 7 (1981).

PRINCIPLE

Reagent for use only in the *in vitro* determination of Prothrombin Time (PT), Activated Partial Thromboplastin time (APTT) and recalcification (plasma clotting) time, as well as in various coagulation factor assays^(1,2,3).

REAGENTS COMPOSITION

Calcium Chloride 0.02 M Liquid preparation of 0.02 mol/L Calcium Chloride and 0.05% sodium azide as preservative.

STORAGE AND STABILITY

 Store at 2-8°C.

The reagent is stable until expiry date printed on the label, when stored in original container.

Reagent deterioration: Presence of turbidity and coloration.

REAGENT PREPARATION

The reagent is ready to use.

PROCEDURE

Refer to your instrument manual, laboratory procedures guide and to the specific test insert for detailed instructions.

NOTES

1. This reagent is NOT FOR USE IN PLATELET AGGREGATION STUDIES, other than for a von Willebrand Factor Assay.

BIBLIOGRAPHY

1. Triplett DA, Heparin: Clinical use and Laboratory Monitoring. In Triplett DA, Laboratory Evaluation of Coagulation, American Society of Clinical Pathologists Press, Chicago p 272 (1982).
2. Harker LA, Hemostasis Manual, FA Davis Co, Philadelphia p 62 (1974).
3. Triplett DA, Harms CS, Procedures for the Coagulation Laboratory, American Society of Clinical Pathologists Press, Chicago, p 7 (1981).

C35104-2/0802
R1.cas/ing